ENGLISH

[Intended use]: The particle Filtering half mask is intended to be worn to protect personnel from the transfer of particulate material. This is a single use, non-sterile product. **[Instruction for use]:** **1.** Wash or disinfect hands before taking out the mask off the package. Avoid touching the inner surface of the mask. **2.** Hold the mask by the ear straps and get the nose and mouth inside the mask. **3.** Fix the ear straps around both ears. **4.** Hold the mask against your chin with the back of one hand. **5.** Place the fingers of both hands in the middle of the nose clip, while pressing inwards. **6.** Move the finger tips along the nose clip to both sides, and press the nose clip into the shape of nose bridge completely. **7.** Do not touch the mask while in use. If so, wash or disinfect your hands. **[Instructions to remove and discard the mask]:** **1.** To remove the mask, use thumbs and index fingers to remove both ear straps **2.** Carefully dispose the used mask adequately **3.** Wash or disinfect hands. **[Precautions]:** **1.** Prior to use please check fit of particle Filtering half mask. **2.** It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face. **3.** This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. **4.** Do not use when concentrations of contaminants are immediately dangerous to health or life. Do not use this product in an explosive atmosphere. **5.** Never alter or modify this mask in any way. **6.** "NR" means Non-Reusable. This mask shall not be used for more than one shift. No maintenance is necessary. Discard mask after use or if damaged in any way. **7.** Children under three years old are not recommended to use the product because of their low vital capacity. **8.** The masks must be stored and transported in their original package and protected by the storage temperature and humidity as suggested by the manufacturer.

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